

## 10 TIPS FOR SUCCESSFUL CONTACT LENS WEAR

Wearing contact lenses provides a great deal of freedom and comfort provided you follow some simple rules of hygiene when handling and caring for your lenses:

- 1 Wash hands and nails, rinse and dry.
- 2 Clean lenses by rubbing between your fingers with a few drops of CAREWAY solution for 20 seconds, then rinse with fresh CAREWAY solution.
- 3 Fill the lens case with fresh CAREWAY solution, soak your lenses overnight.
- 4 Clean the case with CAREWAY solution, rinse and let dry. Keep the case clean and change every 2 months.
- 5 Always follow the instructions for the correct use of CAREWAY solution. Use fresh solution to clean, soak and rinse your lenses.
- 6 Never reuse solution used.
- 7 Remove lenses immediately if you experience red or painful eyes and consult your eye specialist.
- 8 Have an up-to-date pair of glasses for when you remove your lenses.
- 9 Apply makeup after inserting lenses.
- 10 Never use lenses in swimming pools and hot tubs. Do not handle or remove lenses in a public place (except in an emergency). Take your time inserting, removing and cleaning your lenses.

# CAREWAY®

AN ALLIANCE OF EFFECTIVENESS  
AND EXTREME TOLERANCE



Available as Travel kit: 1 x 45ml bottle plus  
1 x mini black pencil eye liner  
Single pack: 1 x 375ml bottle plus  
1 x black pencil eye liner  
Triple pack: 3 x 375ml bottles plus  
1 x black pencil eye liner

MANUFACTURED BY: LABORATOIRES CONTAPHARM, SERVICE  
CLIENTÈLE, BP 16, 77390 VERNEUIL FRANCE  
LABORATOIRES  
CONTAPHARM  
FRANCE  
TEL: +33 (0)1 6425 0333 • FAX: +33 (0)1 6406 3772  
contapharm@contapharm.com • www.contapharm.com

DISTRIBUTED BY: BUTTERFLIES HEALTHCARE, 25A HIGH STREET  
MIDDLETON CHENEY, BANBURY OX17 2PB UK  
TEL: 0845 838 6724 or 0333 700 6724 FAX: 0845 838 6714  
info@careway.co.uk • www.careway.co.uk



# CAREWAY®

AN ALLIANCE OF EFFECTIVENESS  
AND EXTREME TOLERANCE

A MULTIPURPOSE, EXTREME  
TOLERANCE, HYPOALLERGENIC  
CLEANING SOLUTION FOR ALL  
SOFT CONTACT LENSES

LABORATOIRES  
CONTAPHARM  
FRANCE

# CAREWAY®

AN ALLIANCE OF EFFECTIVENESS AND EXTREME TOLERANCE

CAREWAY® multipurpose solution combines efficiency, comfort and safety to perform the functions essential to care for all types of soft contact lenses:

- clean
- moisturise/hydrate
- lubricate

Through its exclusive hypoallergenic formulation, CAREWAY® has the following advantages:

- Ease of use
  - ↳ only one multipurpose product
- Effectiveness thanks to its triple action
  - ↳ cleaning, hydration, lubrication
- Comfort assured throughout the day
- Extreme tolerance
- Free make-up worth at least £6.00



Follow these four simple rules to maintain the health of your eyes and to keep your contact lenses in optimum condition.

**FIRST RULE:** Consult an optometrist regularly to determine the best lenses for your needs and ensure you obtain maximum comfort and visual performance.

**SECOND RULE:** Choose a care solution that provides efficient cleaning, enhanced comfort and ensures safety.

**THIRD RULE:** Follow the advice given on hygiene, care and precautions regarding the wearing and storage of contact lenses.

**FOURTH RULE:** If you wear cosmetics, use a range specially formulated for contact lens wearers.

# CAREWAY®

Women who wear contact lenses are more sensitive to lens care products and makeup (even hypoallergenic ones). This intolerance affects more than one in two women. Contapharm Laboratories, have combined the study of the physiology of the eye and skin, with knowledge of biomaterials to develop EYE CARE COSMETICS. This is a complete range of makeup and skin care specifically for contact lens wearers.



The multi-contoured, extreme tolerance eye pencil from Eye Care Cosmetics is a new development for the health of your eyes. Specially formulated for women who wear contact lenses, it is nickel-free, paraben-free, preservative-free and perfume-free. Being extremely soft it can be used around even the most sensitive eyes. To see the whole range of EYE CARE COSMETICS visit our web site at [www.eyecare-cosmetics.co.uk](http://www.eyecare-cosmetics.co.uk)

AN ALLIANCE OF EFFECTIVENESS  
AND EXTREME TOLERANCE